

## Rx Only

For topical dermatological use only.

**PRODUCT DESCRIPTION:** Eletone® Cream is a nonsteroidal, lipid-rich, fragrance-free emulsion formulated with Hydrolipid Technology for the management and relief of burning, itching, and redness associated with various types of dermatoses. There are no restrictions on age or duration of use and the product has a low potential for irritation.

**INDICATIONS FOR USE:** Eletone Cream is indicated for the management and relief of burning, itching, and redness associated with various types of dermatoses, including atopic dermatitis, allergic contact dermatitis, and radiation dermatitis (post-radiation treatment).

CONTRAINDICATIONS: THIS PRODUCT SHOULD NOT BE USED DURING THE PERIOD OF TIME WHEN RADIATION TREATMENT IS OCCURING BECAUSE OF THE INCREASED RISK OF SKIN TOXICITY WHEN RADIATING THROUGH PETROLATUM AND OIL. Eletone Cream is contraindicated in patients with a known hypersensitivity to any of the components of the formulation.

**PRECAUTIONS:** Eletone Cream is for external use only. Eletone Cream does not contain a sunscreen and should always be used in conjunction with a sunscreen in sun exposed areas.

**INSTRUCTIONS FOR USE:** Apply liberally to the affected areas three times daily or as needed. If skin is broken, cover Eletone Cream with a dressing of choice.

**INGREDIENTS:** Eletone Cream contains petrolatum, purified water, mineral oil, cetostearyl alcohol, ceteth-20, citric acid, sodium citrate, propylparaben, and butylparaben.

## **HOW SUPPLIED:**

Eletone Cream, 100 g tube, NHRIC 0178-0368-01

**STORAGE:** Store at 25°C (77°F); excursions permitted to 15-30°C (59-86°F) [see USP Controlled Room Temperature].

**CAUTION:** Rx only. Federal law restricts this device to sale by or on the order of a physician.



1019